

QUICK GUIDE

mediNiK® for stone removal



TRAINING VIDEO

Further information is available
at www.medinik.de



A PRODUCT BY



FARCO

1. Draw up 1 ml 0.9 % NaCl into an empty syringe
2. Switch off the irrigation
3. K1 syringe (**blue**): Release the pressure point, connect to the working channel and inject under direct vision
4. Carefully add 1 ml 0.9 % NaCl to the working channel
5. K2 syringe (**yellow**): Release the pressure point, connect to the working channel and inject under direct vision
6. Insert the stone basket into the working channel
7. Switch the irrigation back on and start stone removal